

Instruction for use

Plum Wound and Eyewash 0.9% Sodium Chloride

Product list and names

Plum Wound and Eyewash 0.9% Sodium Chloride, 200 ml	Ref. 4691
Plum Wound and Eyewash 0.9% Sodium Chloride, 500 ml	Ref. 4604
Plum Wound and Eyewash 0.9% Sodium Chloride, 1000 ml	Ref. 4707
Plum Wound and Eyewash 0.9% Sodium Chloride DUO, 500 ml	Ref. 4861
Plum Wound and Eyewash 0.9% Sodium Chloride DUO, 1000 ml	Ref. 4800
Plum Wound and Eyewash 0.9% Sodium Chloride QuickRinse, 20 ml	Ref. 5160

Intended purpose

Plum Wound and Eyewash 0.9% Sodium Chloride is a sterile first aid product that rinses and irrigates the dirt, dust, liquid and foreign body from wounds and eyes.


Instruction for use - pictograms - Plum Wound and Eyewash 0.9% Sodium Chloride




Instruction for use - pictograms - Plum Wound and Eyewash 0.9% Sodium Chloride DUO



Instruction for use - pictograms - Plum Wound and Eyewash 0.9% Sodium Chloride QuickRinse

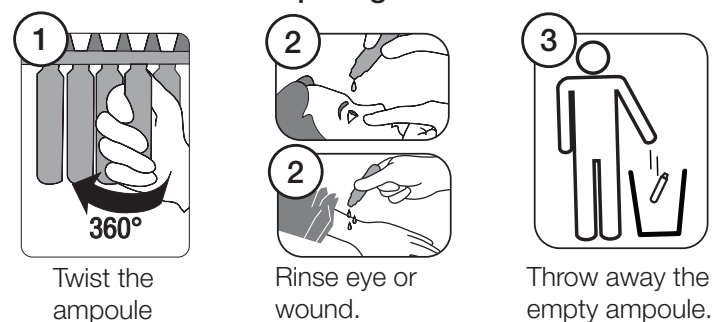

Explanation of symbols

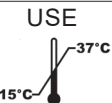	Indicates the medical device manufacturer and is shown together with name and address of manufacturer		Instructs how the bottle must be turned
	CE marking with notified body number		Do not use if package is damaged
	Single sterile barrier system		Do not re-use. The medical device is intended for single use only.
	Medical device		Inform of the manufacturer's batch code to identify the batch / lot.
	Temperature limit for use		Use-by date. Indicates the date after which the medical device must not be used.
	Temperature limit for storage		Production date
	Further information can be found in a booklet on the back of the label		Peel off / open the outer layer of the label to read the booklet with further information

Side effects / malfunction

The products are designed as a first aid product for eye irrigation. If side effects or malfunctions are experienced, please report these to Plum Safety ApS, info@plum.eu or phone +45 6916 9600, so that the necessary remedial measures can be initiated. Serious adverse effects should always be reported to a relevant competent authority.

Contact

Plum Safety ApS
Mandelalleen 1
DK-5610 Assens
+45 6916 9600
Email: info@plum.eu
Web: plum.eu